




EchoTip[®] Ultra

WITH HDFNA[™]




QUICK REFERENCE GUIDE

COMPONENTS

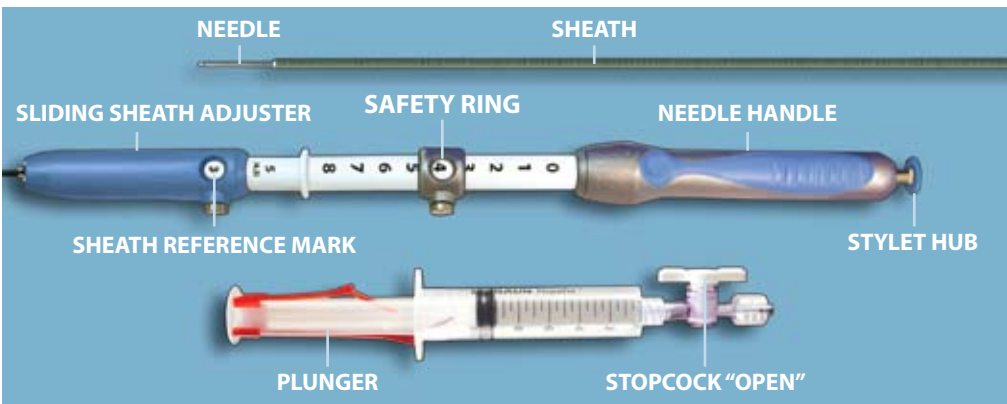


STEP 1

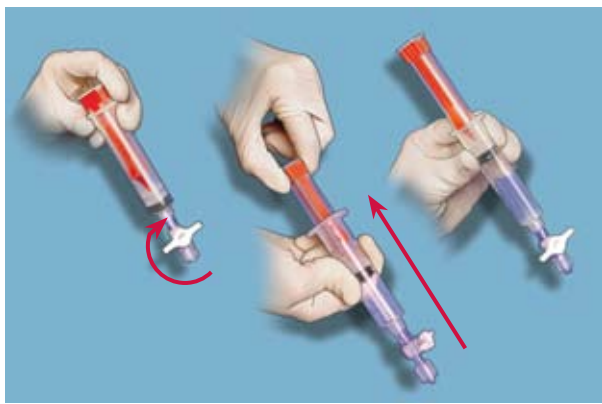


To prepare the aspiration syringe, begin with the plunger in the depressed position. Turn the stopcock to the closed position and pull the plunger back until it is locked into place.

STEP 2

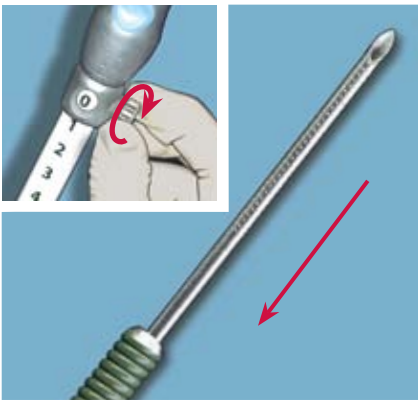

Retract the needle completely within the sheath by locking the safety ring at the "0" needle extension reference mark.

STEP 3

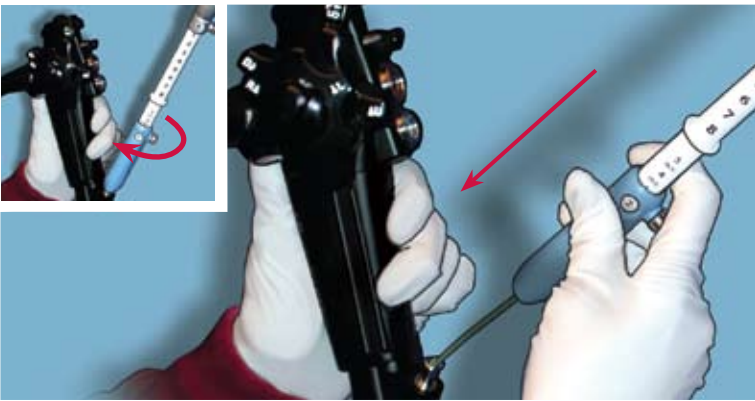


With needle retracted, advance the device into the endoscope and attach the handle to the accessory channel by rotating the device.

STEP 4




To adjust length, loosen the thumbscrew on the sliding sheath adjuster and slide until the preferred length is attained, then tighten.

STEP 5

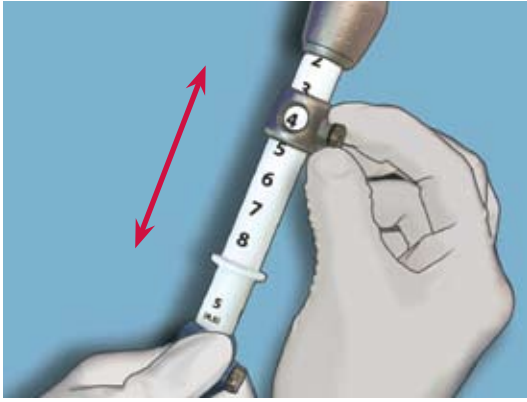

Loosen the safety ring and position at desired needle advancement reference mark and lock in place.

STEP 6

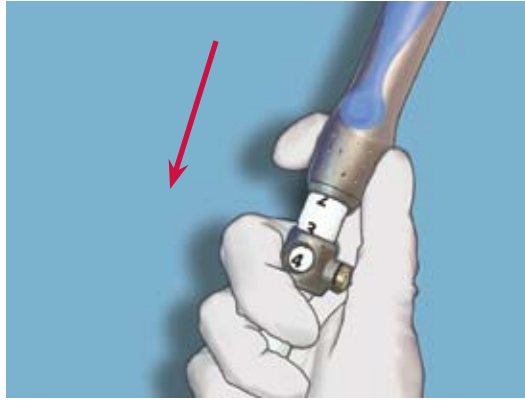


Advance needle into the lesion.




Ultrasound image courtesy of Sergey Kantsevov, MD, Johns Hopkins Hospital, Baltimore, Maryland

STEP 7

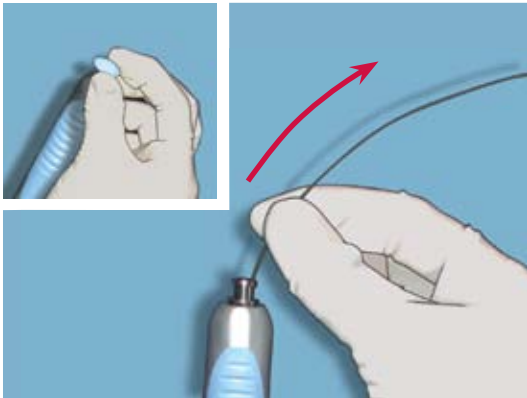


Remove the stylet.

STEP 8

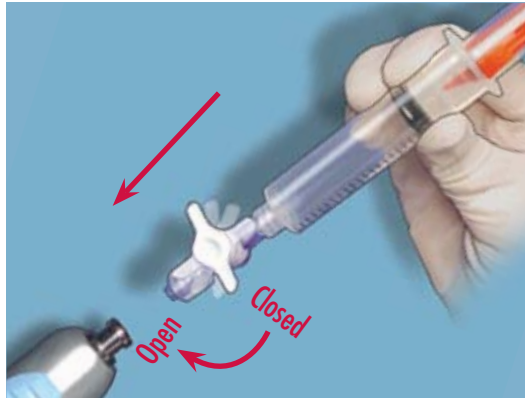


Attach aspiration syringe and open stopcock to begin aspiration. Needle can be moved back and forth to facilitate specimen collection.

STEP 9

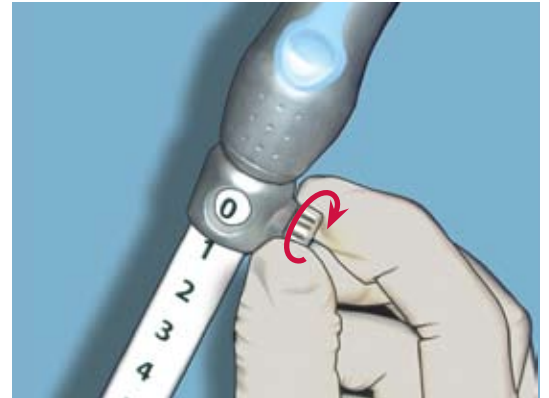

When collection is complete, move stopcock to closed position and retract needle completely. Lock safety ring at the "0" reference mark. Disconnect device from endoscope and remove.

STEP 10

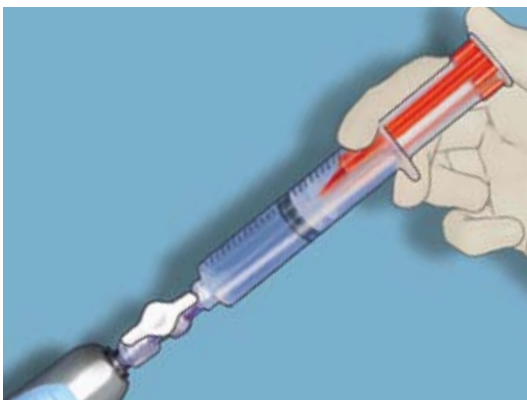

Push the syringe plunger to expel aspirate specimen and prepare per institutional guidelines.

Refer to current instructions for detailed system use.



COOK ENDOSCOPY
4900 Bethania Station Road, Winston-Salem, NC 27105 U.S.A.
Phone: 336 744-0157, Toll Free: (USA) 800 457-4500, Fax: 336 744-5231
EchoTip is a trademark of Cook Urological Inc.
Cook is a registered trademark of Cook Incorporated.
© 2008 Cook Medical

www.cookmedical.com